

# IOW CHLAMYDIA SCREENING PROGRAMME



## Directions on how to use Chlamydia Screening Specimen Kit

If you have not filled in your request form do it now. Please read all the way through these instructions before doing the test.



1) Pee directly into large urine pot (supplied).



2) Open the sealed specimen kit and take out small specimen pot and pipette.



3) Place pipette in your sample contained in the large pot. Squeeze the large end of the pipette filling it with your sample.



4) Place the pipette into the small upright specimen pot and squeeze the large end.



5) Fill the small pot to the top black line as shown.




6) Replace the cap onto the small pot and shake gently a couple of times.



7) Place a numbered sticker from the form onto the small pot.



8) Place the pot into the bag located on the back of the form and seal tightly.

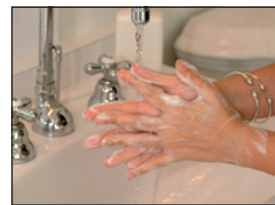

9) Wash your hands thoroughly.



10) Return complete form along with your urine sample to your local doctors surgery or to one of the collection sites listed on your pack.